User Manual: Installation on a Tube - Recommended Installation for Wheeleo®

Caution: If you have a carbon frame, check the maximum torque with the manufacturer. Watch the instruction video at: www.wheebag.com/instruction

Installation Steps:

1. Place the rubber pads:

Insert them correctly into their designated position (*1).

2. Choose the metal band:


Select the short or long band depending on the tube diameter.

3. Attach the band and tension plate:

Install them on the tube following the diagram.

4. Secure and adjust the adapter:

Screw the adapter into place, adjust it to the desired position, and tighten firmly.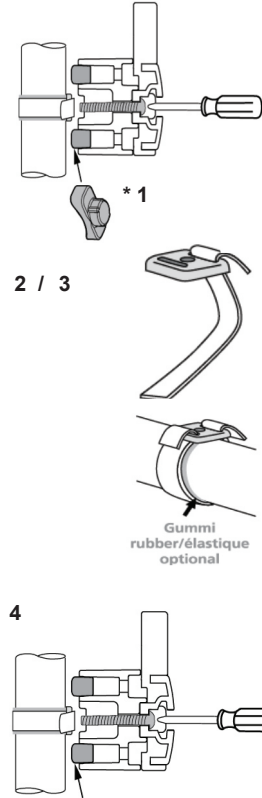

**Your WheeBag® or
accessory is now
ready to be attached!**

User Manual: Installation on a Flat Surface or Panel*

* Only possible if the back of the surface is accessible.

Installation Steps

1. Choose the location:

Identify a suitable spot to attach the adapter. Ensure there is enough space around it to secure your WheeBag® or accessory.

2. Drill the holes:

Drill 2 holes with a 4.5 mm diameter, spaced 26 mm apart (center to center).





3. Attach the base of the adapter:

Use the 2 screws, 2 washers, and round nuts provided. Also position the square tension plate as shown in the diagram.

4. Secure the adapter head:

Screw the head into place in a vertical and straight position.

**Your WheeBag® or
accessory is now ready
to be attached!**

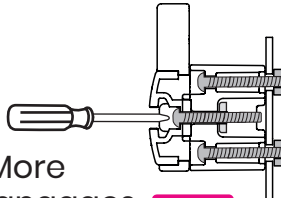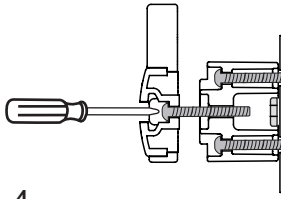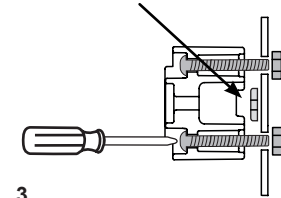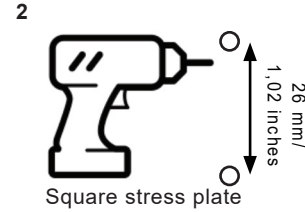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